HydraTherm Deluxe



Thank you for purchasing the HydraTherm Deluxe. This Quick-Start Guide will help you to get started quickly, but does not replace the Operating Manual. Be sure to read, understand, and practice the operating instructions, warnings, and cautions included in the manual.

INITIAL SET-UP

 Clip shelves onto dividers, then place dividers in tank and arrange packs as desired. See illustrations below for recommendation on how to maximize space for use with 12 packs.



 Find drain hose in rear of device. Ensure the valve is closed (turned perpendicular to the hose) before filling the device with water.

FILLING WITH WATER

 To fill the device, first pour in 3 cups of tap water, followed by up to 12 gallons of distilled water, stopping when the water level has reached 2" from the top of the tank.

NOTE: Exact amount of distilled water varies depending on quantity of packs being used.

NOTE: Distilled water helps to maintain a clean device. We highly recommend the use of distilled water. Not using distilled water may cause the unit to require more cleanings or could cause damage to your unit.

Caution

Adding additional packs after unit is filled will cause water to displace and may cause an overflow condition.

POWERING ON THE DEVICE

- Ensure power switch on back of device is in the OFF position.
- Connect the included power plug into the rear of the device, and then into the nearest available outlet.
- Ensure the Heat/Drain switch on the rear of the device is toggled to HEAT.
- Toggle the power switch to the ON position.
 When powered on, the power switch will illuminate, and the digital display on the front of the device will display the current water temperature.

LOW WATER INDICATOR

 Check Temperature Display. If enough tap water was added, the Low Water Indicator should NOT be illuminated at this time. If I ow Water Indicator is illuminated, add more tap water until the indicator light turns off.









Low Water Indicator illuminated Low Water Indicator not illuminated

TEMPERATURE SETTING

- Do not exceed 140° F (60° C) for best therapeutic effect, patient comfort, and HydraHeat Pack longevity.
- It can take several hours for the water to reach the Set Temperature. The default Set Temperature is 135°F (57°C), to change the Set Temperature refer to instructions below.

To enter Temperature Set Mode:

- Press SET button
- Press UP or DOWN arrows until desired temperature is reached (Note: The temperature cannot be set higher than 140°F / 60°C).

- Press SET button again to lock in temperature.
- The unit will begin heating the water.









WARNING: The scalding temperature of water is 140° F (60° C). Never place hands into the scalding water. Use plastic-coated tongs to retrieve hot pack if necessary.

HYDRAHEAT PACKS

- · HydraHeat Packs absorb heat, not water, and provide therapeutic heat up to 45 minutes.
- Placing a towel or pillowcase between between HydraHeat Pack and patient is recommended.
- Terry covers are not recommended when using HydraHeat Packs.

Have guestions? Contact Compass Health Brands:

Phone: 888.549.4945

Email: customerservice@richmarweb.com



Compass Health Brands Corp. Toll Free 1.888.549.4945 richmarweb com

Made in Mexico